




Gynatrof®

(Hyaluronic Acid and Vitamin E)


After use, clean the applicator thoroughly as follows:

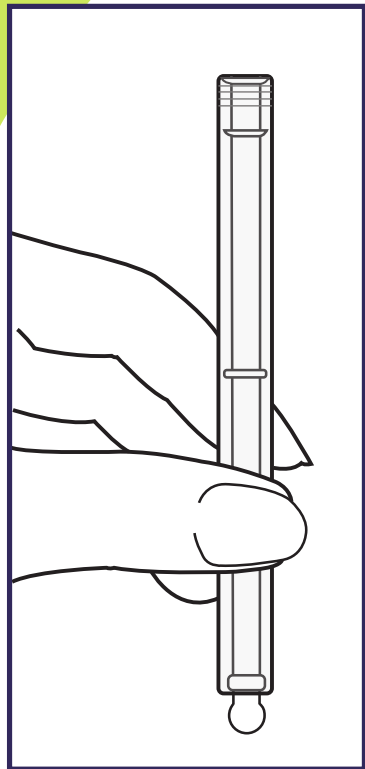


To dismantle the applicator, press the white round tip of the piston down against any hard surface until it clicks to release.

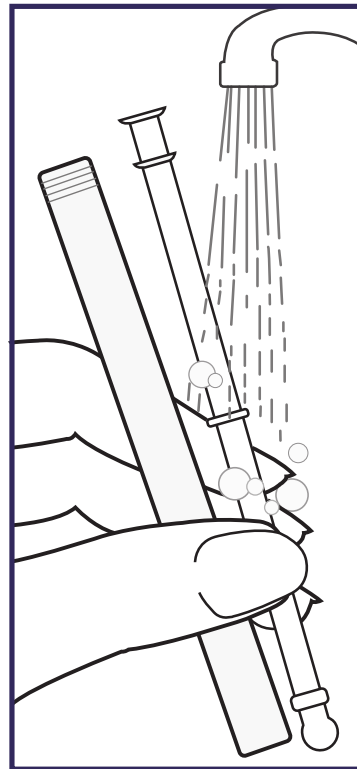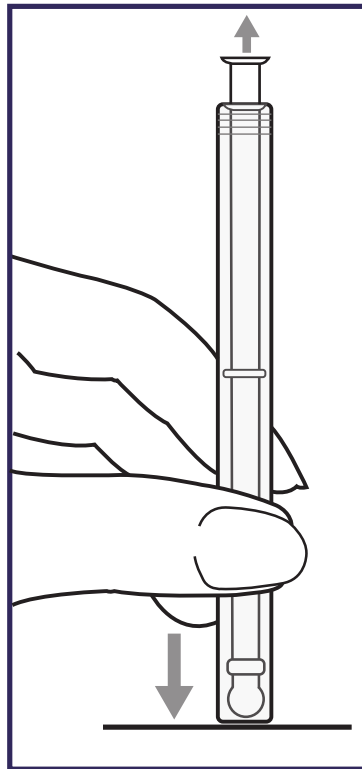

Remove the piston from the cylinder and wash with lukewarm water: rub with fingers to remove any excess gel, fill and drain the cylinder repeatedly with water. Shake the cylinder to remove excess water and dry with a clean towel.

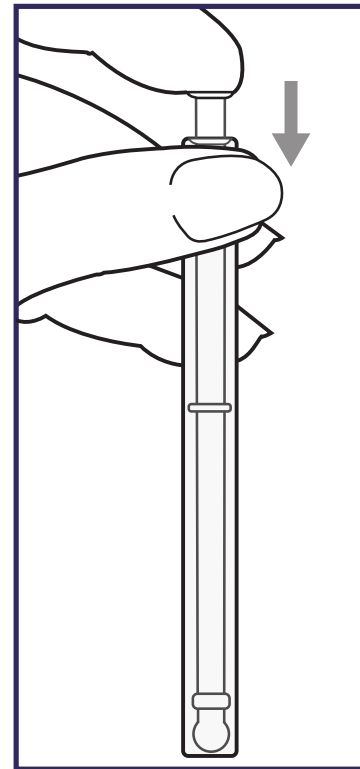

Store the dismantled applicator in the original carton box until next use. To reassemble the applicator, insert the thin end of the piston into the wider threaded end of the cylinder and with light pressure, click the piston back into place.

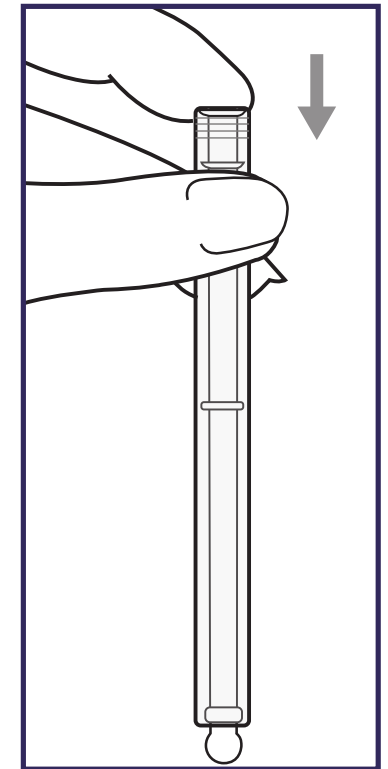

For more information: www.gynatrof.com

Imported by: Tyros Biopharma Inc., 100, boul. Alexis Nihon, #596
Montreal, Quebec H4M 2P1

Last revision: October 2018

TYROS
BIOPHARMA