



# Gynatrof®

(Hyaluronic Acid & Vitamin E)


**To ensure proper hygiene,  
please follow these steps to clean the applicator after each use:**

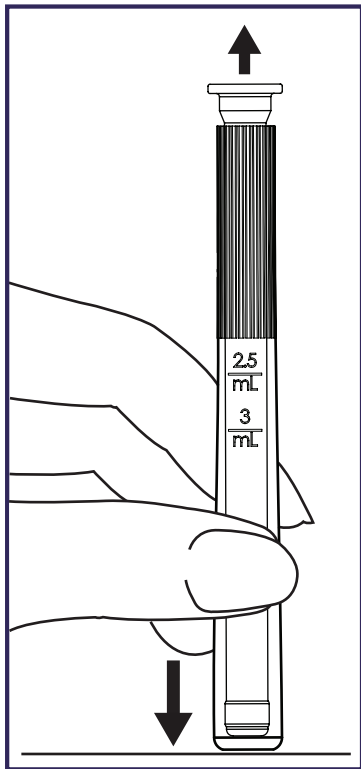

Disassemble the applicator by gently pulling apart the piston from the device.

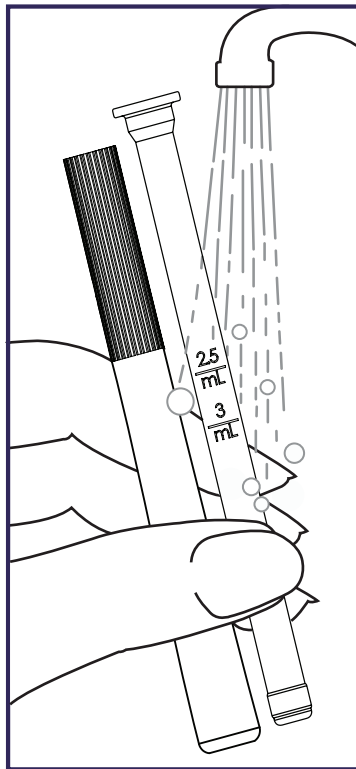

Clean it by washing with mild soap and lukewarm water. Rub the applicator with your fingers and fill and drain the cylinder several times until all soap residue is thoroughly removed. Shake the applicator to remove excess water and then dry it with a clean towel.

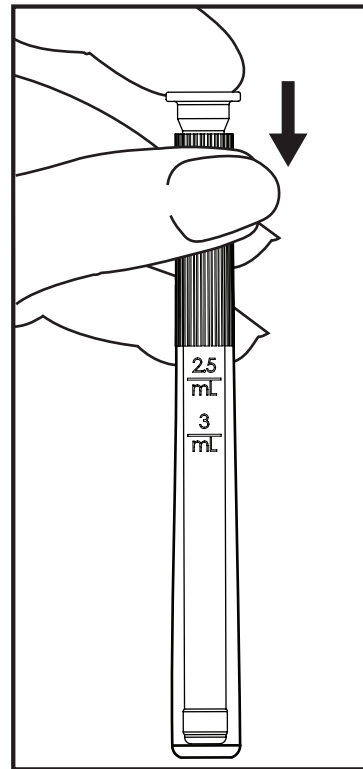

To maintain cleanliness, store the disassembled applicator in its original carton box and reassemble it when you plan to use it next.

For more information: [www.gynatrof.com](http://www.gynatrof.com)

Manufactured by: Tyros Biopharma Inc.  
4702 Louis-B.-Mayer, # 305  
Laval, Québec H7P 0L9

**TYROS**  
BIOPHARMA